



Viking XS / Viking S

Quick Reference Guide


English

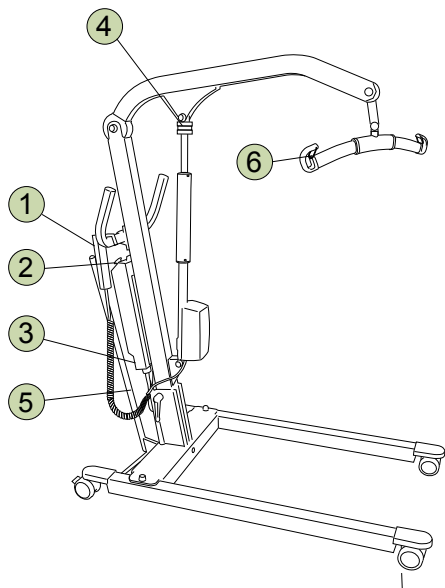

1 Hand Control

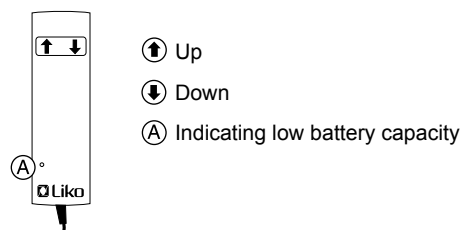


5 Base-Width Adjustment

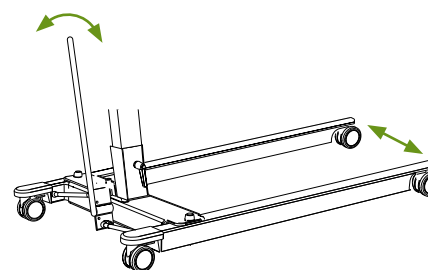

2 Emergency Stop


Activate:

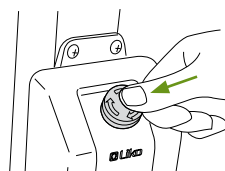


Reset:




6 Latches

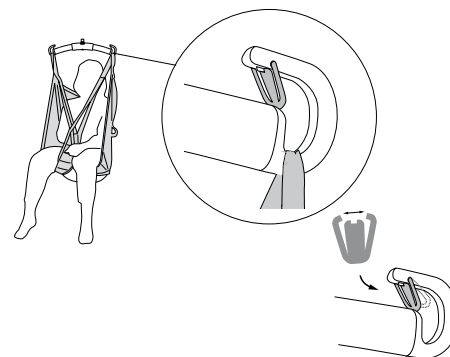


3 Electrical Emergency Lowering

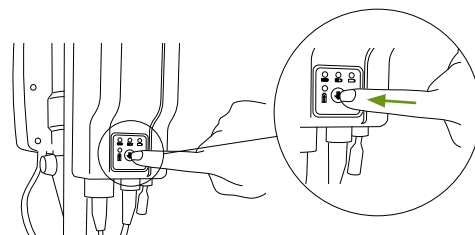

4 Mechanical Emergency Lowering

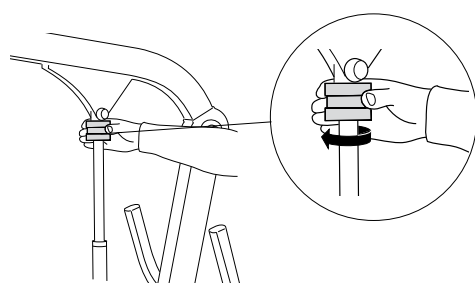


Charging the Batteries

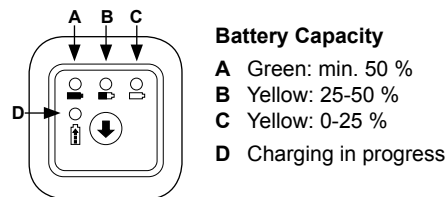

Battery Capacity

- A Green: min. 50 %
- B Yellow: 25-50 %
- C Yellow: 0-25 %
- D Charging in progress

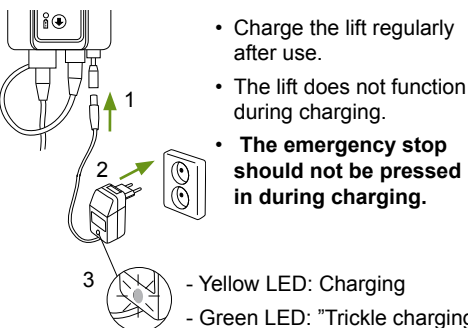

- Charge the lift regularly after use.
- The lift does not function during charging.
- **The emergency stop should not be pressed in during charging.**

- Yellow LED: Charging
- Green LED: "Trickle charging"

Ergonomics

- Never lift manually – let the lift unit do the lifting.
- Take it easy. Work calmly and methodically.
- Don't strain your back! Work in a walking stance with your back straight.
- Use the bed's raising and lowering functions.
- Reduce the lever effect. Work close to the patient and the lift.

Cleaning

When necessary, clean the lift with a moist cloth, using warm water or disinfectant.

Simple Troubleshooting

Check the following:

- the emergency stop is not pressed in
- the battery is charged
- the hand control is properly connected.

Other Quick Reference Guides

Quick reference guides for Liko's lifts and slings are available for downloading at www.liko.com.

© Copyright Liko AB



A Hill-Rom Company

www.liko.com

Manufacturer:

Liko AB
SE-975 92 Luleå
Sweden
info@liko.se

Safety Instructions

- Read the instruction guides for the lift and lifting accessories before use.
- Plan the lifting operation carefully.
- Check to ensure that the sling's strap loops are correctly fastened to the sling bar hooks when the sling strap is extended, but before the patient is lifted from the underlying surface.
- Ensure that the personnel using the equipment have received appropriate instructions and training.
- **Maximum load: 160 kg (350 lbs.)**

To retain max. load it is essential to use accessories rated for the same max. load or greater.

IMPORTANT!

This quick reference guide does not replace the lift's instruction guide, which can be downloaded from www.liko.com.

900003033

7EN138502-01